



Guide for Uploading Ultrasound Scans Using SonoClipShare

Introduction

This guide is designed for individuals on a remote mentoring package with g&h Remote Mentor. Here we provide a detailed, step-by-step process to upload and share ultrasound scans using SonoClipShare. The platform serves as a secure, cloud-based repository and teaching aid for ultrasound education.

Step-by-Step Guide

- 1. Exporting Scans:**
 - Begin by exporting your ultrasound scans from the ultrasound machine to a USB drive. Ensure that the scans are in a supported format (e.g., mp4 for videos, jpg for still images).
- 2. Creating an Account:**
 - Visit [SonoClipShare](https://www.sonoclipshare.com) and create an account if you do not already have one. Follow the prompts to register using your preferred email for verification.
- 3. Uploading Scans to Your Archive:**
 - Log into your account and select the option to create a new archive.
 - Connect your USB drive to your computer and upload the scans to the newly created archive.
 - If your scans are not already anonymised, you can download the de-identifier tool which is free to download using the link: <https://www.sonoclipshare.com/tools.php>
- 4. Organising and Tagging Scans:**
 - Once your scans are uploaded, you can organise them within the archive. Use tags (e.g., #Cardiac, #Abdominal) to categorise and describe each scan, which aids with future searching.
- 5. Sharing with Educators or Peers:**

- Share your archive with your designated remote mentor, details of this will be provided upon purchasing a remote mentoring package.

6. **Receiving Feedback:**

- Remote mentors will view your scans and leave comments directly on the clips or stills where necessary. Check the 'Shared with Me' section regularly for feedback and discussions.

7. **Continued Learning and Collaboration:**

- Engage in discussions through comments to deepen understanding and interpretation skills. Utilise feedback to improve your scanning and diagnostic techniques.

Technical Support and Resources

For any technical issues or further guidance on using the SonoClipShare platform, refer to the wiki section on the website or contact the support team. Link 📌

<https://wiki.sonoclipshare.com>

Conclusion

This guide should serve as a comprehensive introduction to uploading and sharing ultrasound scans using SonoClipShare. It is designed to facilitate seamless and efficient learning and collaboration among medical professionals.

By following these steps, you can efficiently integrate SonoClipShare into your educational or professional practice, enhancing both learning outcomes and collaborative opportunities.

Any questions? Email me at: ghremotementor@gmail.com



g&h CritCareEcho

High quality echo and ultrasound education